



AUSTRALIAN MOUTHGUARDS™

Safety and how to use our Polish

We apply our mouthguard polish to medical gauze. By rubbing all over the guard the texture of the gauze combined with the polish is enough to further buff and smooth the material. We then freshen up the gauze with polish wiping over leaving a nice even glossy look. Being extremely careful not to overheat we then use a hot air gun to wave over the guard a couple of times.

Hot air guns can be bought from most hardware stores.

Flaming may cause carbon discoloration of the mouthguard so is not recommended.

ATTENTION when using our mouthguard polish always

Avoid contact with skin and wear suitable gloves.

Or use Derma Shield skin protection as per instructions.

First Aid: eye contact—Irrigate with water and seek medical advice.

Spillage: Absorb with sand or granules.

Storage: Maximum temperature 40c

Fire: Use dry powder or CO2 as extinguishing agents.

Harmful: May cause lung damage if swallowed.

If swallowed do not induce vomiting; seek medical advice immediately.

Do not breathe fumes.